

COMPONENTS




Erect-D™ VACUUM Erection System is a medically-proven high quality penis pump, which is a non-invasive, safe and effective solution to follow a *Sexual Fitness* routine and induce on demand drug-free instant erections as a solution to *Erectile Dysfunction*.

Economically designed and engineered, Erection System consists of a cylinder that fits over the penis, with an electronic motor to create suction and induce a vacuum environment around the penis which draws blood into it and helps the penis to become engorged.

Note:

Charged your **Erect-D™** before using (See diagram)


Erect-D™ Vacuum Pump

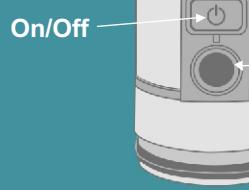

On/Off


vacuum release button

On Charging Red light
Full charge Blue light

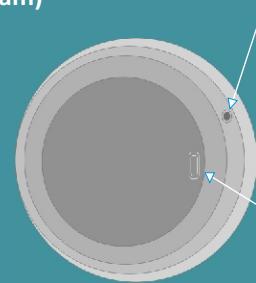

Charging port

Charging Cable

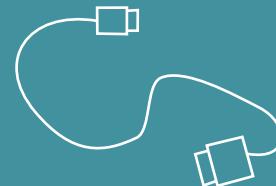


Note: Read instructions carefully before using



If instructions are not followed, Erection System will not work.


Adapter: Custom Fitting Adapter Inserts for different penis sizes, to be placed in the penile tube after the ring loading on penile tube then cone is removed.

Put a little amount of lubricant around the base of the penis, the shaft, and glans (see figure).

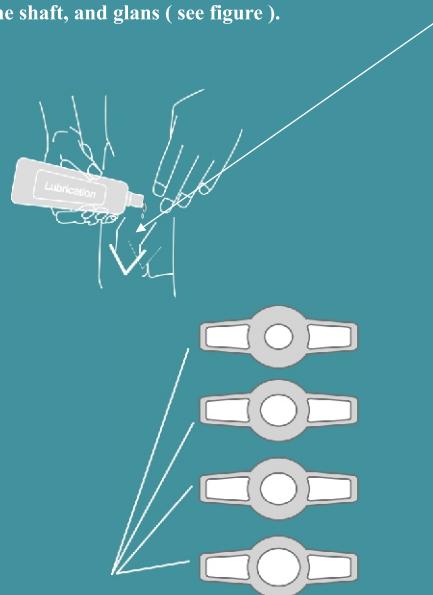

ON/OFF button

Erect-D™ Vacuum Motor

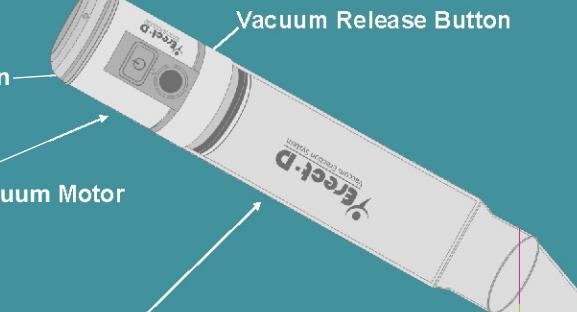

Vacuum Release Button

Medical Grade Cylinder

Easy Loading Conical System fits into penile tube to assist in placing ring on penile tube



Lubricant
for personal use

Erect-D™ Erection Assistant Rings / Constriction Rings:

to help maintain the erection. For different penile thicknesses