

# INSTRUCTIONS FOR ERECTILE DYSFUNCTION THERAPY

Vacuum Erection System

1. Lubricate your penis, the inside of the cylinder and pubic hair area (the area that comes in contact with the cushioning rings), with the sealing gel. This helps form a good vacuum seal at your body and prevents the penis from sticking to the walls of the cylinder.

2. Insert the penis into the cylinder and start the vacuum process. Operate the motor by depressing the activation button. Depress the button for approximately 10 seconds, that should create a seal between the system and your body. Continue pressing the button until you have developed a sufficient vacuum. Use the least amount of vacuum necessary to obtain an erection.

3. As soon as the desired erection is achieved, transfer the Erection Assistant Ring from the cylinder onto the base of the penis with your fingertips. Make sure the Erection Assistant Ring is transferred all the way around. Any vacuum that is left can be released by depressing the manual vacuum release button. Practice a few times without the Erection Assistant Ring.

Note 1. After you have been using the system for 30 to 60 days, you may want to recheck the sizing of the rings to be used. For example, you may find that you are now able to use the next size larger rings. using the larger ring may increase comfort.


Note 2. Some couples may prefer to stop intercourse at least once and reuse the **ED Vacuum**.

Note 3. If a condom is desired it should be applied after an erection is formed.

Note 4. Following use, the Erection Assistant Ring, the Cushioning Ring and Cylinder can be washed with soap and water, dried and stored. Make sure the Erection Assistant Ring bag is air free and tightly sealed for longer life. **DO NOT EVER PUT THE MOTOR IN WATER.**

**ATTENTION**  
When placing the Cylinder on the pump head, **DO NOT OVER TIGHTEN**. Twist one quarter (1/4) turn to tighten. Twisting too hard or too far will break your system. If this happens it **WILL NOT** be covered under warranty!


Slide the Erection Assistant Ring throughout to the Easy Loading Conical System and on to the end of the Cylinder. Also lubricant on top of the ring Loading Cone.

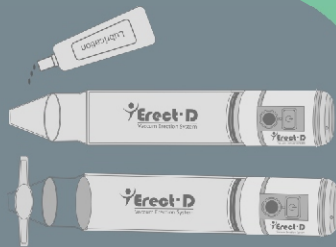

Insert your penis into the prepared Cylinder. if you feel any pain press vacuum release button immediately the pain will disappear.

Activation button

Remove the Easy Loading Conical System and load the Connector Rings.

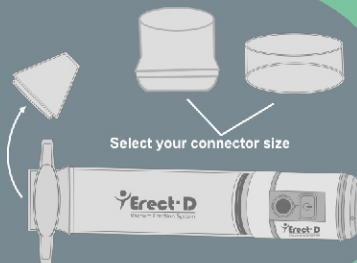

Select your connector size



Start the vacuum process with activation button. Once erection is complete slide the ring on to base of the penis once ring transfer is completely, you are ready.

Activation button

Lubricate your penis, with your pubic hair area and corner inside of the Cylinder, see diagram.



When you finish sexual intercourse, remove the rubber to make the penis flaccid as before

Remove the device by pressing push release vacuum button. When you finish intercourse remove the ring to make the penis flaccid as before.